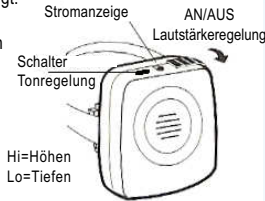
D **BENUTZUNG**

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GB

Note: users of digital hearing aids must make sure their aids have been programmed for use with the 'T' or 'Loop' setting. The Amplifier functions as an inductive coupler with the Tcoil to minimize the background noise and thus improves the clarity of reception. If you experience feedback (a high pitched squeal) you can correct this by reducing the volume of the amplifier. To avoid distortion , do not use the CLA20 with a telephone which already has extra receiving amplification facility .

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GEEMARC
CLEAR SOUND
HEAR THE DIFFERENCE

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GB

- Volume control (0 to +20dB)
- Tone control (Bass/Normal/Treble)
- Strap for easy mounting on your telephone handset
- Soft carrying case
- 2 AAA batteries included

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
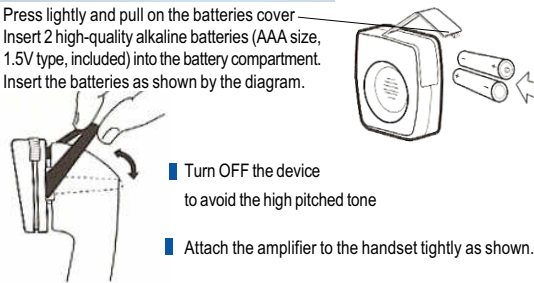
GB

USING

- Press lightly and pull on the batteries cover. Insert 2 high-quality alkaline batteries (AAA size, 1.5V type, included) into the battery compartment. Insert the batteries as shown by the diagram.
- Turn OFF the device to avoid the high pitched tone
- Attach the amplifier to the handset tightly as shown.

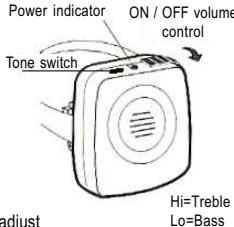- You can increase the volume and adjust the receiving tone according to your own hearing requirements. Turn ON the amplifier. The power indicator will light up. Adjust the volume to suit your hearing.
- Use the Tone switch (Bass/Normal/Treble) to adjust the frequency response according to your own hearing requirements.

- If the voice sounds low or distorted or if the power indicator is low or fails to light up, the batteries may be low. Check and replace the batteries.
- Turn OFF the amplifier at the end of the call.

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